


## Importing Printing Profiles (.ini) for Asiga Printers


Visit ApplyLabWork [Support](#) page and download .ini file for the material you are using.

**Note,** Some browsers will require confirmation. Please proceed with the download.

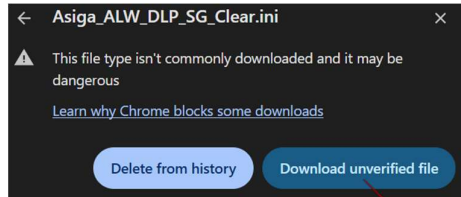



Check: Composer version 2.1.4 or later

Start Composer, click **[Options]** → **[Preferences]**


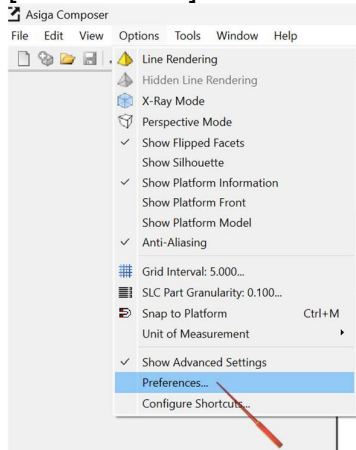


Find **[Materials]** tab, and click the plus icon . Select the folder where the .ini files locate.

**[Select Folder]** → **[OK]**

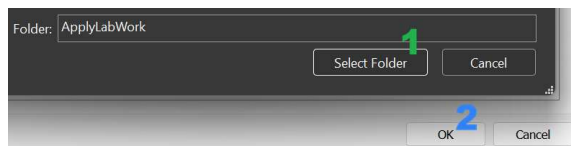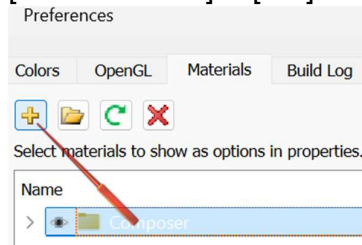

Click **[NEW]**, Using drop-down menu to select **[Select Material]**, **[Select Layer thickness]**, and **[build tray]**.

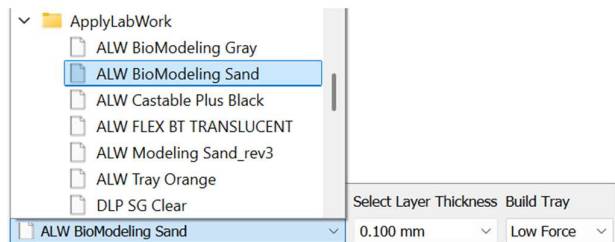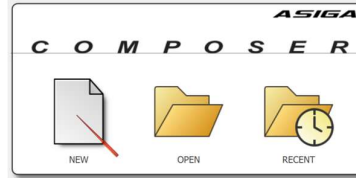

Enjoy printing with ApplyLabWork materials !

## Importing Printer Setting