


## Importing Printing Profiles(.cfgx) in CHITUBOX


Visit ApplyLabWork [Support](#) page and download .cfgx file for the printer and material you are using.

Check: CHITUBOX version 1.0.1 or later


If the printer is not specified, click the printer+ icon next to [printer]

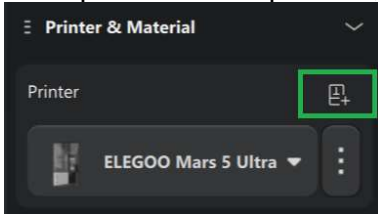


Find the printer model, click **[Next]**, and jump to Step 2.

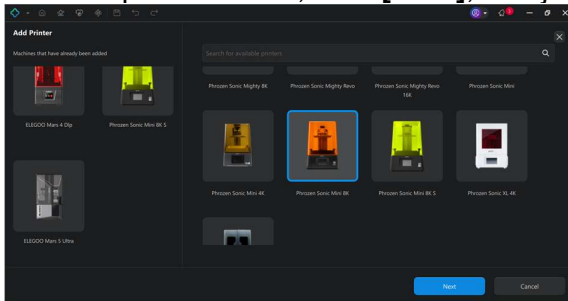


Step 1. Start CHITUBOX, and click the plus icon next to [Material Configuration]

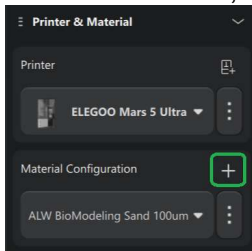

Step 2. Click import icon in the task bar . Then, select the downloaded file, and click **[Open]**

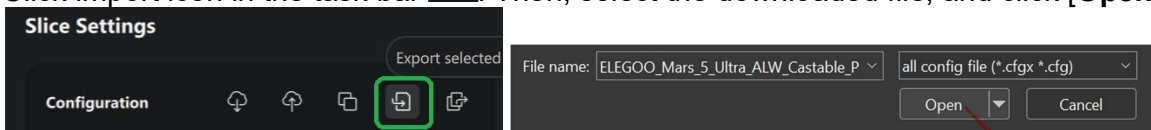

Step 3. The pre-defined settings will appear in material list. Then, click **[Apply to Project]**

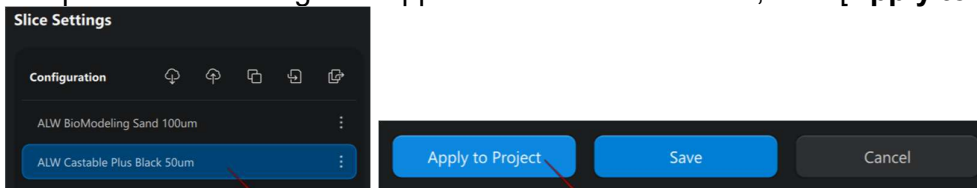

Note: Each printer is unique, and light intensity can vary even among identical models. Minor adjustments may be necessary, especially true for open-system MSLA printers.

Enjoy printing with ApplyLabWork materials!

### Importing Printer Setting